

## Installing & using IFSYS-8003 on PCs with Windows Vista operating system

---

### **Advisory for Cozmo pump users**


If you encounter problems using your Cozmo pump with the CozManager software and you are running Windows Vista or later versions of Windows, please upgrade your CozManager software to CozManager 2.0 version. For more details about CozManager 2.0, please proceed to Smiths Medical website – <http://www.cozmore.com/about-cozmanager/>, or contact Smiths Medical for further information.

The following provides you a step-by-step guide on how to install and use the IFSYS-8003 adapter on a PC that is running on Windows Vista operating system.

Please refer to the following steps:

- Step 1 :** Connect the IFSYS-8003 adapter into a USB port on the PC.
- Step 2 :** Windows Vista automatically detects the adapter as a SigmaTel STIR-4200 adapter and loads the driver onto the PC. To check if the IFSYS-8003 has been properly installed, please do the following:



**2.1** Under Control Panel of the PC, click on Device Manager as shown below

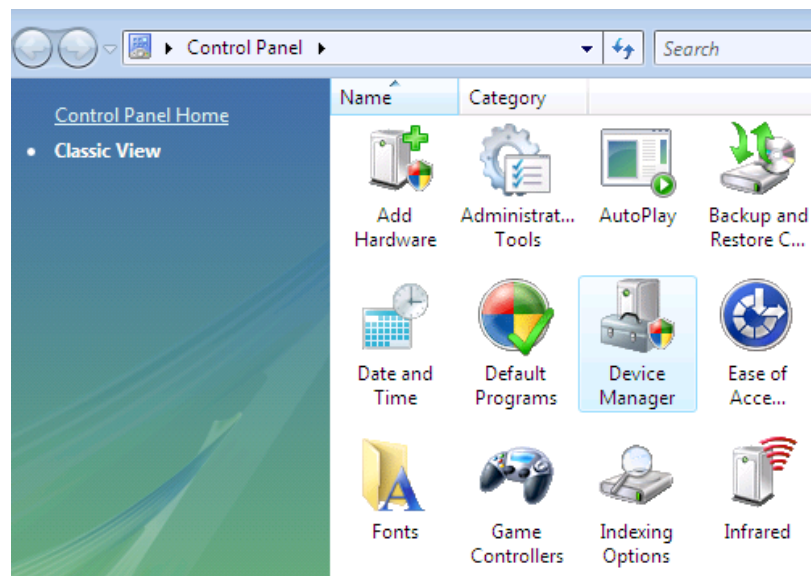

**2.2** Under Device Manager, click on Infrared devices. Under Infrared devices, you should see SigmaTel USB-IR Dongle as an Infrared device. This shows that the driver has been loaded.

## Installing & using IFSYS-8003 on PCs with Windows Vista operating system

---

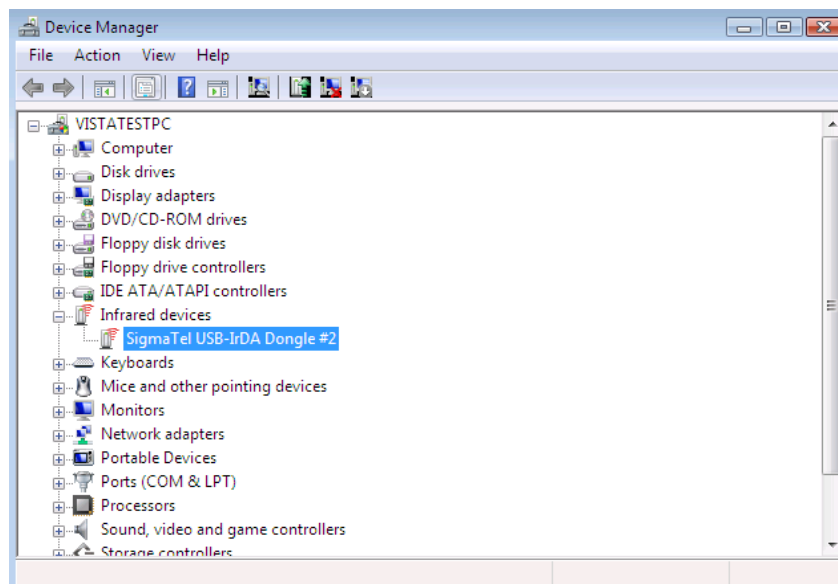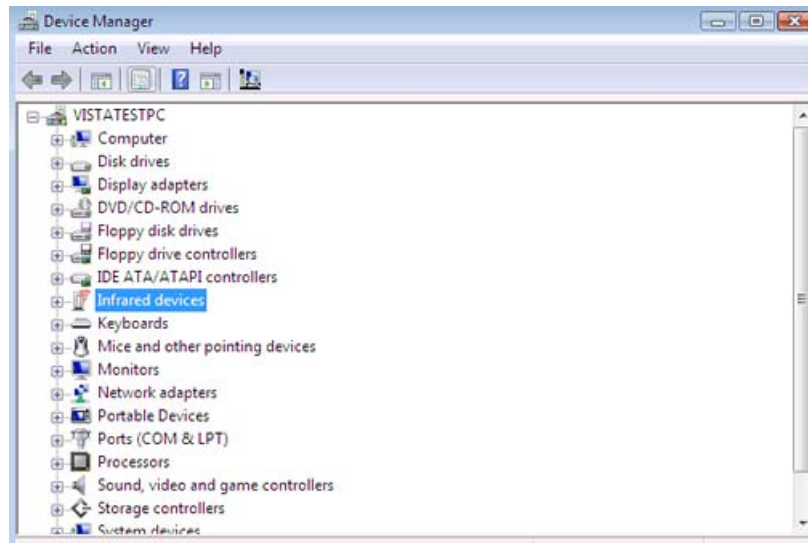

**Step 3 :** Check if the red LED on the IFSYS-8003 adapter is flashing a few times every 3 seconds (Note: Please inform us it doesn't).


**Step 4:** Try to transfer a file, using the IFSYS-8003 adapter between the PC and an IR-enabled mobile device such as a mobile phone, a Palm, etc.

### 4.1 To transfer a file from a mobile device to a PC

**4.1.1** Turn on the Infrared feature on the mobile device and place the mobile device's Infrared port towards the "eye" of the IFSYS-8003 adapter. Windows Vista should inform you that a remote device has been detected. You should also see a "Send files to another computer" icon on the desktop as shown below

## Installing & using IFSYS-8003 on PCs with Windows Vista operating system


---

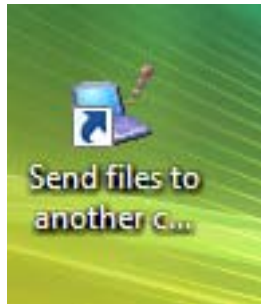

**AND** an InfraRed icon at the bottom-right corner of your desktop as shown below.




**4.1.2** Send a file from the mobile device via Infrared. The PC will detect that a device is trying to send a file to it as shown below.

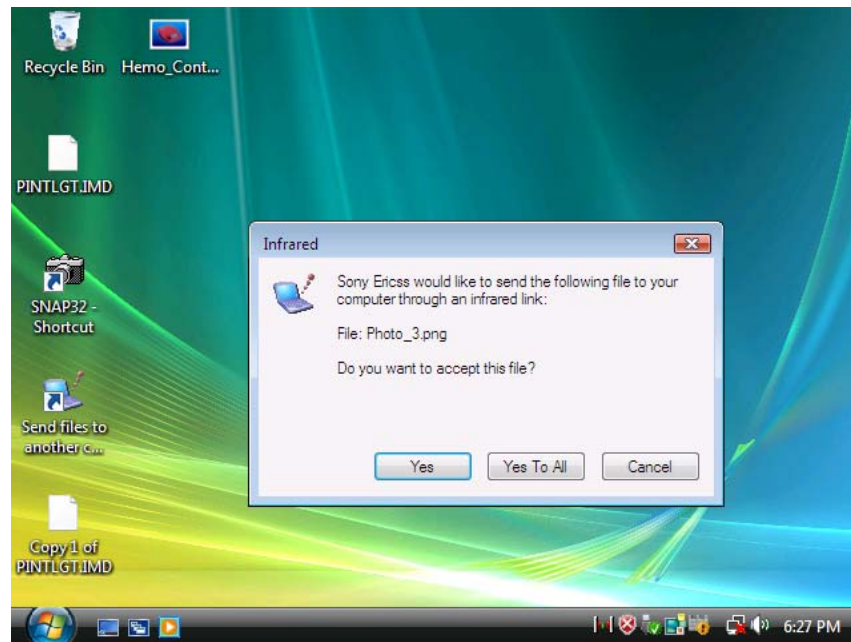

**4.1.3** Click Yes and the file will be sent from the mobile device to the PC. If the file has been successfully received by the PC, you should see the following screen.

## Installing & using IFSYS-8003 on PCs with Windows Vista operating system


---

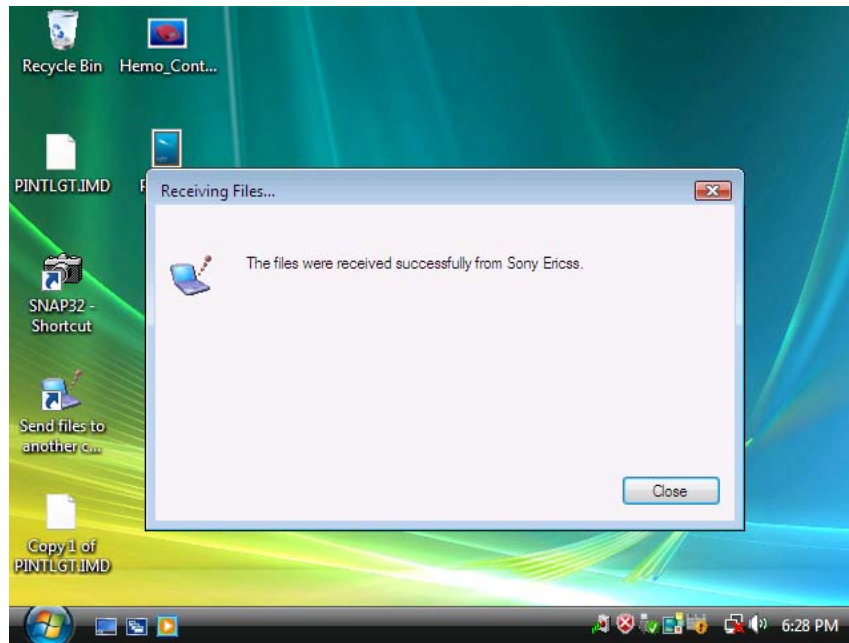

### 4.2 To transfer a file from a PC to a mobile device

4.2.1 Repeat Step 4.1.1 above.

4.2.2 Click on the “Send files to another computer” icon on the desktop as shown below

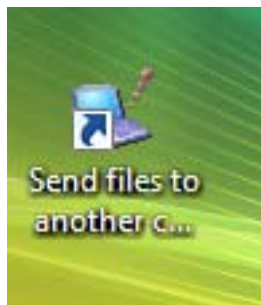


**OR** the InfraRed icon at the bottom-right corner of your desktop as shown below.




The following window will open.

## Installing & using IFSYS-8003 on PCs with Windows Vista operating system

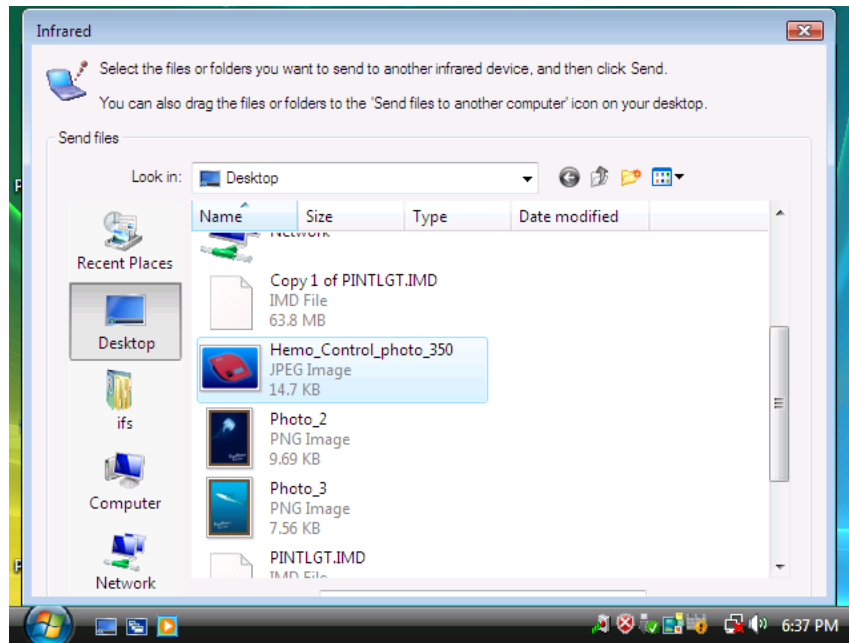

**4.2.3** Search for the file you want to transfer and click on it. The following window should appear, informing you that the PC is connected to your mobile device and it's transferring your selected file. Once the file has been successfully transferred, the "Sending files" window will close automatically.

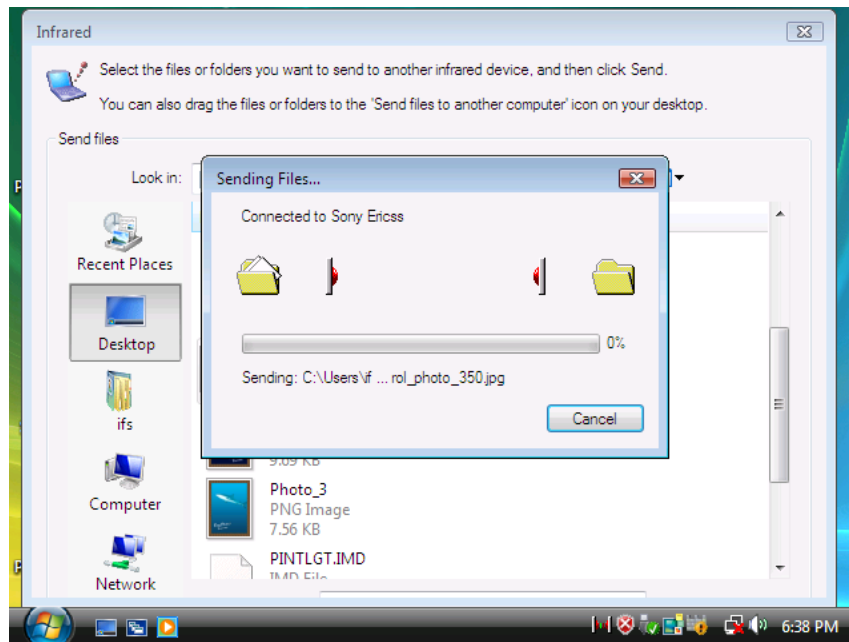

**Step 5:** Congratulations! You have successfully installed and used the IFSYS-8003 adapter on your Windows Vista PC.